

Wondfo SARS-CoV-2 Antibody Test Procedure

Allow the device, buffer and specimen to room temperature (10° C - 30° C)

Remove the test cassette from the foil

Place test cassette on a level surface

Add 1 drop (10µl) of whole blood or serum or plasma into small well (sample well)

Add 2-3 drops (80µl) of buffer solution to the large well (buffer well)

Wait for 15 minutes and read the result.

Positive result: Colored bands appear at both test line (T) and control line (C)

Negative result: Colored band appear at control line (C) only.

Arora
92/22